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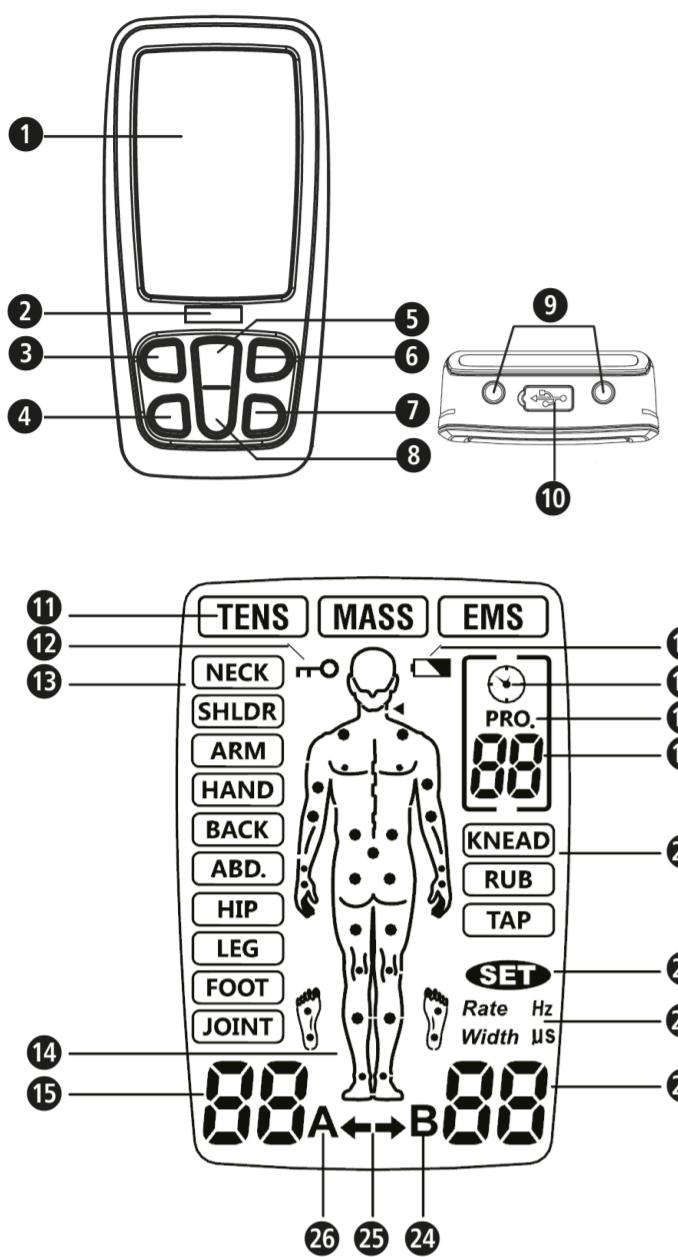
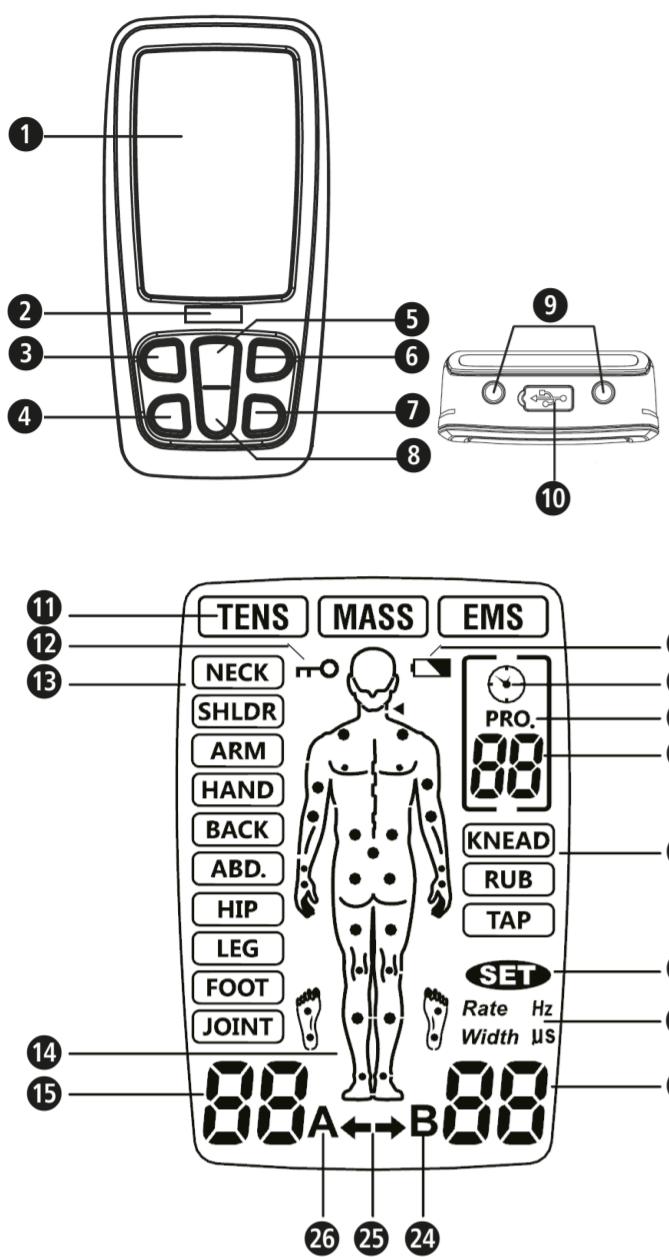
GB Instructions for use  
3-in-1-Electrotherapy device TT 205

## Device and controls

## Explanation of symbols

**IMPORTANT**  
Non-observance of these instructions can result in serious injury or damage to the device.

**WARNING**  
These warning notes must be observed to prevent any injury to the user.

**CAUTION**  
These notes must be observed to prevent any damage to the device.

**NOTE**  
These notes give you useful additional information on the installation or operation.

Device classification: Type BF

LOT

IP22 Protection rating against foreign objects and water

EC REP Authorized representative in the EC

Manufacturer

Date of manufacture

## GB Safety instructions

Please read these instructions for use carefully before using the device, in particular the safety instructions. Keep the instructions in a safe place for future reference. If you give the device to someone else, be sure to include these instructions for use.

## Intended use

The medisana 3-in-1 Electrotherapy device TT 205 is intended for the following areas of application:

- TENS mode: Temporary relief of muscle pain in the neck, shoulder, back, joints, hip, hand, abdomen, foot, upper (arm) and lower (leg) extremities due to exercise or normal house-work.
- EMS mode: The EMS stimulation program stimulates healthy muscles to improve and promote muscle performance.
- Massage mode: The massage program ensures muscle relaxation through vibration to loosen tense muscles.

## Contraindications

- Do not use the device if you have a pacemaker, defibrillator, or other electronic or metallic implants in your body. These could result in electric shock, burns, electrical interference or death.
- Do not use the device if there are cancerous or other lesions in the treatment area.
- Do not treat any part of the body that has swelling, burns, inflammation, rashes (e.g. phlebitis, thrombophlebitis, varicose veins, etc.), wounds or sensitive areas.
- The electrodes must not be attached so that current can flow through the carotid sinus region (front throat) or transcerbral (through the head).
- This device should not be used on excessively stressed skin areas.
- The device must not be used if there is an inguinal hernia.
- Do not treat scarred parts of the body after surgery - wait at least 10 months after surgery before using the device.
- Do not use the device in the case of severe circulation problems in the lower limbs.

- Therapeutic sessions with the medisana TT 205 do not replace medical diagnosis or treatment. Ask your doctor about each kind of pain or disease, before using the device.
- If your pain does not diminish, does not become significantly milder or last longer than 5 days, discontinue use and contact your doctor.
- Do not apply stimulation over your front and lateral neck because this could cause severe muscle spasms resulting in closure of your airway, difficulty in breathing or adverse effects on heart rhythm or blood pressure.
- Do not apply stimulation across your chest because the application of electrical current to the chest may cause rhythm disturbances to your heart, which could be lethal.
- Do not apply stimulation over - or in proximity to - cancerous lesions.
- Do not apply stimulation in the presence of electronic monitoring equipment (e.g., cardiac monitors, ECG alarms), which may not operate properly when electrical stimulation device is in use.
- Do not apply stimulation when in bath or shower.
- Do not apply stimulation while sleeping.
- Do not apply stimulation while driving, operating machinery or during any activity when electrical stimulation can put you at risk of injury.
- Apply stimulation only to normal, intact, clean, healthy skin.

- The long-term effects of electrical stimulation are unknown. Electrical stimulation device cannot replace drugs.
- Stimulation should not take place while the user is connected to high-frequency surgical equipment, which may cause burn injuries on the skin under the electrodes, as well as problems with the stimulator.
- Do not use the device in the vicinity of short or microwave therapeutic devices as they may affect the output power of the device.
- Never use it near the cardiac area. Stimulation electrodes should never be placed anywhere on the front of the thorax (marked by ribs and breastbone), but above all not on the two large pectoral muscles. There it can increase the risk of ventricular fibrillation and lead to cardiac arrest.

- Never use it on the eye, head and face area.
- Never use it near the genitals.
- Never use it on the areas of the skin which lack normal sensation.

- Keep electrodes separated during treatment. It could result in improper stimulation or skin burns if electrodes are in contact with each other.
- Keep the stimulator out of reach of children.
- The appliance is intended for use by adults only.

- The therapy should be pleasant. If you feel pain or feel the application is unpleasant, do not increase the intensity but stop the usage and consult your doctor.

- TENS is not effective for pain of central origin including headache. It is not a substitute for pain medications and other pain management therapies.

- TENS is a symptomatic treatment and, as such, suppresses the sensation of pain that would otherwise serve as a protective mechanism.

- Effectiveness is highly dependent upon the patient selection for a practitioner qualified in the management of pain patients.

- Since the effects of stimulation of the brain are unknown, stimulation should not be applied across your head, and electrodes should not be placed on opposite sides of your head.

- The safety of electrical stimulation during pregnancy has not been established.

- You may experience skin irritation or hypersensitivity due to the electrical stimulation or electrical conductive medium (silica gel).

- If you have suspected or diagnosed heart disease or epilepsy, you should follow precautions recommended by your physician.

- Use with caution if you have a tendency to bleed internally, e.g. following an injury of fracture or during your menstrual period.

- Consult with your physician prior to use the device after a recent surgical procedure, because stimulation may disrupt the healing process.

- For single patient use only.

- This stimulator should not be used by patients who are noncompliant and emotionally disturbed including dementia.

- Only use the device as described in this instruction manual. Any other use will be deemed as not in accordance with the instructions and may lead to damage to property or even injury to persons. medisana assumes no liability for damages incurred as the result of improper use. The device is designed for private use.

- Rare cases of skin irritation may occur at the site of the electrode placement following long-term application.

- Do not use this device in the presence of other equipment which sends electrical pulses to your body.

- Do not use sharp objects such as a pencil or ballpoint tip to operate the buttons on the control panel.

- Check the electrode connections before each use.

- Electrical stimulators should be used only with the electrodes recommended for use by the manufacturer.

## Side effects

- Skin irritation or burns can occur in the area of the electrode application.
- In very rare cases, first-time users of an EMS device have reported drowsiness or fainting. We recommend that you use the device while sitting until you get used to its use.
- If you feel uncomfortable, reduce the stimulation intensity to a comfortable level. Contact your doctor if the problems persist.

## Scope of delivery

First, check whether the device is complete and is not damaged. If in doubt, do not operate the device and contact your supplier or service centre.

The contents include:

- 1 medisana 3-in-1-Electrotherapy device TT 205
- 4 Electrodes 50x50 mm
- 2 Connection cable
- 1 USB cable
- 1 Instructions for use

If you notice any transport damage while unpacking, please contact your supplier immediately.

**WARNING**  
Keep packaging away from children!  
Choking hazard!

## Device and controls

- ① Display field
- ② Charge indicator
- ③ On/Off/M button
- ④ P button
- ⑤ + key
- ⑥ B button
- ⑦ CH button
- ⑧ - button
- ⑨ Output port
- ⑩ USB port
- ⑪ Therapy mode
- ⑫ Device locked
- ⑬ Body region
- ⑭ Recommended treatment points
- ⑮ Intensity Channel A
- ⑯ Battery low
- ⑰ Timer symbol
- ⑱ Program symbol
- ⑲ Program no. / Therapy time
- ⑳ Massage mode
- ⑳ SET symbol
- ㉑ Frequency/Pulse width
- ㉒ Intensity Channel B
- ㉓ Symbol for Channel A
- ㉔ Channel indicator
- ㉕ Symbol for Channel B

## Charge battery

Disconnect the connection cables from the output connections ⑨ (if necessary).

The device cannot be used while charging.

Connect the USB cable to the USB port ⑩ on the device and a suitable 5V USB charging adapter.

The charging indicator ② lights up yellow while the device is being charged.

When charging is complete, the ② charging indicator will light up green. Charging can take up to 2 hours.

## Cable connection

Make sure that the device is switched off before you connect the cable. First connect the electrodes with the two connection cables and then the cables with the device. On the upper side of the device there are two output connections ⑨ for connecting the electrode lines. The device has two connections for two channels (A and B). Depending on your needs, you can decide whether you want to use only one channel with one pair of electrodes or connect both channels with two pairs of electrodes. With two pairs of electrodes you can stimulate two different body regions at the same time.

**WARNING**  
NEVER insert the electrode cables into an electrical outlet or any other power source!

## Applying the electrodes

The reusable carrier film can be removed immediately before use - keep it for later use. Then place the electrodes according to your pain area (follow the instructions of your doctor).

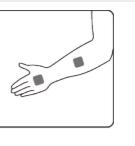
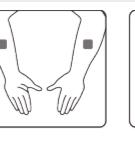
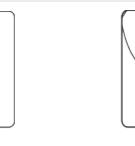
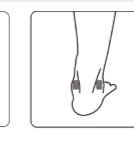
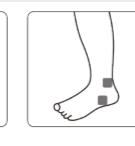
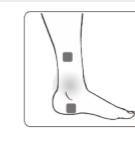
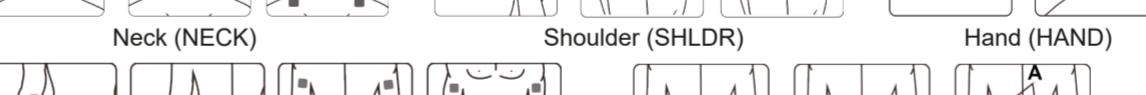
- The device must be switched off when applying the electrodes!
- Remove any skin cream or ointment residue before applying the electrodes. The skin must be fat-free and dry.
- Do not stick the electrodes on injured or inflamed skin areas (wounds, pimples, rashes, redness, etc.).
- Always apply the electrodes according to your doctor's instructions and only then switch on the device.
- Make sure that the electrodes cannot fall off during the therapy. This can lead to an unintended electric shock!
- Switch off the device immediately if an electrode detaches from the skin during treatment.

In the following you will find placement suggestions for the electrodes depending on the therapy mode. However, please note that each patient reacts differently to stimulation current therapies - therefore the positioning of the electrodes and the program selection for your requirements may differ from the therapies presented here. You should make a note of the optimum parameters for you (electrode position, program and current) for subsequent treatments.

The recommended treatment points ⑭ for each program are shown in the illustration. The following illustrations show an example of how to attach the electrodes to these treatment points.



## Examples of electrode positioning in the EMS therapy mode:

Ankle (JOINT)

Wrist (JOINT)

## Examples of electrode positioning in the EMS therapy mode:



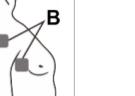
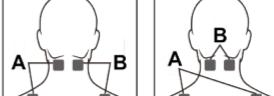
Neck (NECK)

Shoulder (SHLDR)

Hand (HAND)

Arm (ARM)

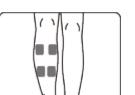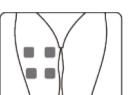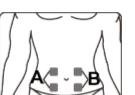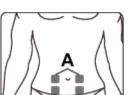Back (BACK)




Abdomen (ABD.)

Legs (LEG)



Hip (HIP)

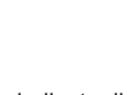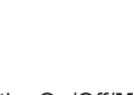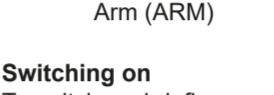Foot (FOOT)

Knee (JOINT)

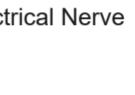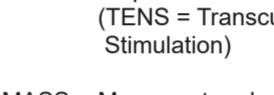Elbow (JOINT)

Elbow (JOINT)

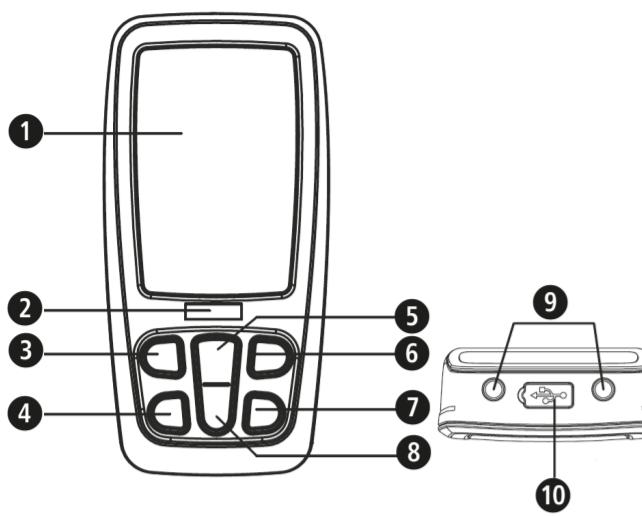
Knee (JOINT)

Hand (HAND)


Arm (ARM)

Shoulder (SHLDR)

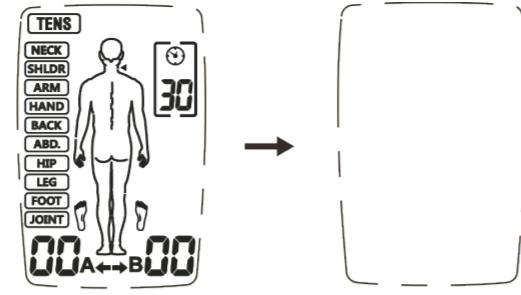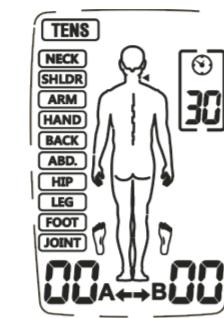
**GB Instructions for use  
3-in-1-Electrotherapy device TT 205**
**Device and controls**

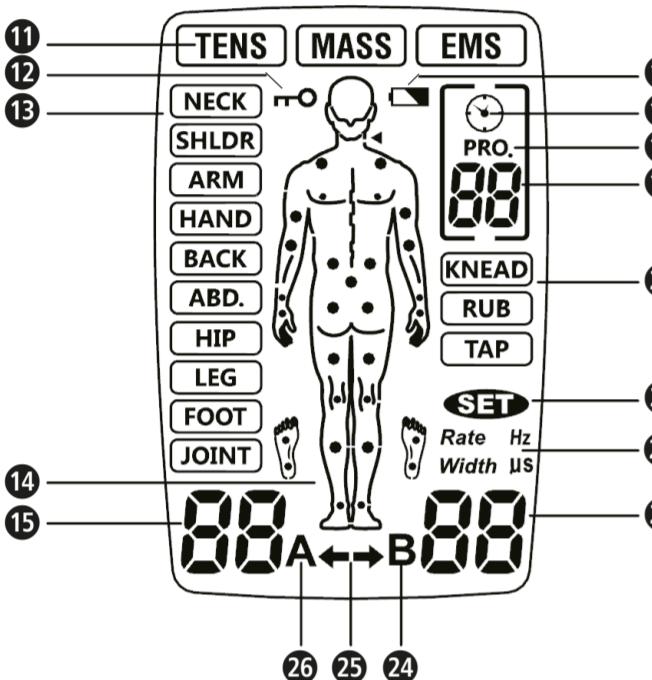
Press the On/Off/M button ③ to exit the application. Pressing the On/Off/M button ③ again for a long time switches the device off.

The device has an automatic load detection at all set intensities above level 5. If the electrodes do not stick to the skin properly or there is another connection problem, the device automatically switches to level 0 and the "A" or "B" symbol flashes. The device then switches back to standby mode.

If the battery voltage is too low, the "Battery low" symbol ⑩ flashes. Stop the application and replace the batteries.

**Explanation of symbols**

Recycling symbols/codes:  
These are used to provide information about the material and its proper use and recycling.

MD Medical device

Temperature range

Ambient pressure limitation

Humidity range

**Use of the electrodes**

- The electrodes supplied may only be used with the medisana 3-in-1 Electrotherapy device TT 205. Make sure that the device is switched off before applying and removing the electrodes.
- If you want to reposition the electrodes during use, switch off the device first.
- Use of the electrodes can cause skin irritation. If you should notice this (e.g. red skin, blistering or itching) stop the use. Do not use the electrodes regularly on the same skin areas.
- Never use the electrodes together with other people.
- The electrodes must be in full contact with the skin to avoid possible "hot spots", which can cause burns.
- Do not use the same electrodes more than 10 times because the connection to the skin surface deteriorates with each use.
- The adhesive force of the electrodes depends on the skin properties, the storage condition and the number of applications. If your electrode pads no longer stick completely to the surface of the skin, replace them with new ones. After use, stick the electrode pads back onto the protective film and keep the electrodes in the storage bag so that they do not dry out. As a result, the adhesive force is retained over a longer period of time.
- Clean and dry the skin areas where the electrodes are to be applied.
- Never remove the electrodes from the skin while the device is still in operation.
- Only use original electrodes that are recommended by the manufacturer. Other electrodes could cause injuries.
- Each person reacts differently to electrical nerve stimulation. There may be individual deviations in the correct placement of the electrodes. Contact your doctor to find out which placement is best for you.
- Do not use electrodes smaller than the ones provided. Such use could make the current density too high and cause injury.
- Do not change the electrode pads supplied, e.g. by cutting.
- Make sure that the area from which the pain originates is completely covered with the electrode pads. In the case of painful muscle groups, all affected muscles must be reached by the electrodes.

**NOTES on the TENS application**

- If you find the output intensity too strong, you can reduce it with the "-" button ⑧;
- As long as you feel comfortable during the application, continue it to the end. Pain improvement usually occurs after about 5 to 10 minutes;
- We recommend 1 to 2 applications per day for about 1 week;
- If there is no significant improvement in pain after this time, contact your doctor.

**NOTES on the EMS application**

- Pay attention to the correct placement of the electrodes as described in these instructions;
- We recommend 1 to 2 applications per day for about 1 week;
- If you feel uncomfortable during the application, pause the application and/or reduce the intensity.

**Maintenance and cleaning**

In order to ensure trouble-free and long-term functional reliability of the device, the following points should be observed:

- Remove the electrodes from the device and clean the device with a soft, slightly damp cloth. In the case of heavy soiling, you can also use a mild detergent.
- Protect the device from moisture. Do not hold the device under running water and do not immerse it in water or other liquids.
- Do not place the device on hot surfaces and do not expose it to direct sunlight.
- Clean the surface of the electrode pads with a slightly damp cloth. Make sure that the device has been switched off beforehand!
- For hygienic reasons, each user should use their own electrode pads.
- Do not use aggressive substances or chemical cleaners to clean the device.
- Make sure that no liquid gets into the device. Do not use the device again until it is completely dry.
- Never clean the device during operation. Before cleaning, make sure that the device is switched off and the batteries have been removed!

**Maintenance**

- The manufacturer cannot be held responsible for damage caused by unauthorised repairs. If your device has a technical problem, contact customer service.
- Do not try to carry out repairs yourself.
- Opening the device will render the warranty void.
- Your device was tested before being sold and does not require calibration or regular maintenance. If you have any questions, please contact customer service.

**Troubleshooting**

Problem	Possible causes	Countermeasures
No display after changing the battery	Batteries not inserted correctly or foreign objects in the battery compartment. Battery type may be wrong.	1. Check and clean the battery compartment. 2. Insert new batteries of the correct type. Pay attention to the correct polarity!
No or too weak stimulation	- Electrodes not placed correctly on the skin - Connection to the device is not ideal - Used batteries - Too dry skin	1. Check that the electrodes are correctly attached to the skin and that the cable connection to the device is optimum. 2. Replace the battery.
The skin tickles or burns during use	- The application is too long. - Electrodes are not placed correctly on the skin or the contact is not optimum. - The skin is hypersensitive	1. Use the device once a day and shorten the time of use. 2. Check that the electrodes are correctly attached to the skin. 3. Wipe your skin and the electrodes with a damp cotton cloth. 4. Consult your doctor if you have hypersensitive skin.
The device stops during the application	- Electrodes are not placed correctly on the skin or the contact is not optimum - Used batteries	1. Check that the electrodes are correctly attached to the skin. 2. Replace the battery.

**Program tables**

Mode	Body part	Program	Frequency Hz	Impulse width $\mu$ s	Duration min.	Waveform
TENS	Neck	01	80-120	120-100	Standard: 30 Adjustable: 5-90	Modulation
		02	4	150-200		Modulation
		U1	Standard: 35 Adjustable: 2-100	Standard: 200 Adjustable: 100-300		Consistent
	Shoulder	01	80-100	100		Modulation
		02	2-60	260-160		Modulation
		U1	Standard: 100 Adjustable: 2-100	Standard: 150 Adjustable: 100-300		Short Pulse groups
	Arm	01	2	250		Consistent
		02	100	150		Short Pulse groups
		U1	Standard: 100 Adjustable: 2-100	Standard: 200 Adjustable: 100-300		Consistent
	Hand	01	100	100		Consistent
		02	2-10	200		Modulation
		U1	Standard: 60 Adjustable: 2-100	Standard: 260 Adjustable: 100-300		Modulation
	Back	01	60/80/45/10/50/35	200		Modulation
		02	6/8/10	250		Modulation
		U1	Standard: 55 Adjustable: 2-100	Standard: 200 Adjustable: 100-300		Modulation
	Abdomen	01	80-120	120-100		Modulation
		02	120	55		Consistent
		U1	Standard: 80 Adjustable: 2-100	Standard: 100 Adjustable: 100-300		Consistent
	Hips	01	100	150		Short Pulse groups
		02	40/6/50	200		Modulation
		U1	Standard: 80 Adjustable: 2-100	Standard: 180 Adjustable: 100-300		Consistent
	Leg	01	40/6/50	250		Modulation
		02	80	150		Modulation
		U1	Standard: 6-10 Adjustable: 2-100	Standard: 200 Adjustable: 100-300		Modulation
	Foot	01	80-120	100-120		Modulation
		02	2-10	200		Modulation
		U1	Standard: 2-60 Adjustable: 2-100	Standard: 260-160 Adjustable: 100-300		Modulation
	Joints	01	100	150		Short Pulse groups
		02	120	100-120		Modulation
		U1	Standard: 80 Adjustable: 2-100	Standard: 180 Adjustable: 100-300		Consistent
EMS	Neck	01	30	200		Synchronous
		02	40	200		Synchronous
		U1	Standard: 50 Adjustable: 20-100	Standard: 200 Adjustable: 100-300		Synchronous
	Shoulder	01	45	200		Synchronous
		02	55	200		Synchronous
		U1	Standard: 80 Adjustable: 20-100	Standard: 200 Adjustable: 100-300		Synchronous
	Arm	01	50	150		Synchronous
		02	60	150		Synchronous
		U1	Standard: 80 Adjustable: 20-100	Standard: 150 Adjustable: 100-300		Synchronous
	Hand	01	4	200		Synchronous
		02	5	300		Synchronous
		U1	Standard: 20 Adjustable: 20-100	Standard: 150 Adjustable: 100-300		Synchronous
	Back	01	60	200		Synchronous
		02	70	200		Synchronous
		U1	Standard: 80 Adjustable: 20-100	Standard: 200 Adjustable: 100-300		Synchronous
	Abdomen	01	20	200		Synchronous
		02	50	200		Synchronous
		U1	Standard: 60 Adjustable: 20-100	Standard: 200 Adjustable: 100-300		Synchronous
	Hips	01	30	150		Synchronous
		02	60	150		Synchronous
		U1	Standard: 40 Adjustable: 20-100	Standard: 150 Adjustable: 100-300		Synchronous
	Leg	01	20	200		Synchronous
		02	80	200		Synchronous
		U1	Standard: 25 Adjustable: 20-100	Standard: 200 Adjustable: 100-300		Synchronous
	Foot	01	4	200		Synchronous
		02	5	300		Synchronous
		U1	Standard: 20 Adjustable: 20-100	Standard: 200 Adjustable: 100-300		Synchronous
Massage	Knead	01	28-44	120-250	30	Modulation
	Rub	01	25-79	120-250	30	Modulation
	Tap	01	49-97	100-240	30	Modulation

**Technical information**

Name and model:	medisana 3-in-1-Electrotherapy device TT 205
Model no.	R-C4A
Power supply:	Li-Ion battery, 3.7 V

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